

Size: 236x66.5mm

C100 M95.7 Y32 K23

Tear at notch

MEDSOURCE

MS-23970

Silicone Nasopharyngeal Airway

SIZE: 20FR

Sterile

Not made with natural rubber latex

Directions:
1. Lubricate the airway with water soluble lubricant.
2. Gently insert airway into nostril.
3. Do not force. Gently rotate the airway to ease entry.

CAUTION:

Federal (USA) law restricts this device to sale by or on order of a physician.

Replace per CDC guidelines and/or hospital policy.

For single use only. Do not resterilize.

(01) 0 0840155 40547 9

LOT

XXXXXXX

XXXX-XX-XX

Single Use Only

Manufactured for:

8600 Shelby Court

Chanhassen MN, 55317 USA

MedSource

www.medsourcelabs.com

Made in China

REV. 01

236mm

66.5mm