

# MEDSOURCE

www.medsourcelabs.com




## Infant

Not made with  
natural rubber latex.  
SINGLE USE ONLY

### Medium Concentration Mask with Universal Connector MS-26043-U

DISPOSABLE: FOR SINGLE PATIENT USE. NOT INTENDED FOR REPROCESSING.

**⚠ CAUTION:** Federal law (USA) restricts this device for sale by or on the order of a physician.

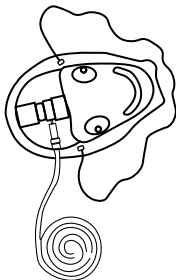

(01) 0 0840155 40043 6

LOT



REV. 07

## Directions for Use:

**NOTE:** These directions are general guidelines intended for use by qualified medical personnel.

1. Attach the oxygen supply tubing to the gas source and set the oxygen to the prescribed flow.
2. Check for gas flow through the device.
3. Place the mask on the patient's face with the elastic strap below the ears and around the neck.
4. Gently pull the ends of the strap until the mask is secure.
5. Mold the metal strip on the mask to fit the face.



Manufactured for:  
MedSource  
8600 Shelby Court  
Chanhassen, MN 55317 USA  
[www.medsourcelabs.com](http://www.medsourcelabs.com)  
Made in China



**WARNING:** This product can expose you to chemicals including Di(2-ethylhexyl)phthalate (DEHP), which is known to the State of California to cause cancer and birth defects or other reproductive harm. For more information go to [www.P65Warnings.ca.gov](http://www.P65Warnings.ca.gov).